

SURGISPON[®] Dental

Absorbable Haemostatic Gelatin Sponge



- **Fast and effective**
- **Absorbs 40-50 times of its own weight**
- **Saves valuable surgical and anaesthetic time**



SURGISPON®

SURGISPON® is a sterile hemostatic absorbable gelatin sponge made of pure pharmaceutical gelatine (lyophilized hydrolyzed collagen) foam, which gives it a uniform porosity & reacts neutrally. The dried foam is cut, packaged and sterilised using gamma radiation. SURGISPON® is immediately available for use in the operating theatre and does not require any sterilisation by dry heat or autoclaving.

HOW to use SURGISPON® Dental

- Soak the cube in sterile saline solution or use dry.
- If used dry, the sponge is slightly compressed. The sponges must be applied to a bleeding area under light pressure for one or two minutes until the bleeding stops.
- If used wet, withdraw SURGISPON® cubes and squeeze thoroughly between gloved fingers to expel air bubbles, replace in saline and keep there until needed.
- When bleeding is controlled, the cubes can be left in situ.


SURGISPON® may be used in combination with antibiotics, Thrombin, chemo therapeutics without reduction of the sponges haemostatic effect. Carefully read applicable instructions for use before usages.

Products




WHERE to use SURGISPON® Dental

To achieve immediate hemostasis in:-
Extractions & Oral bone and soft tissue surgery

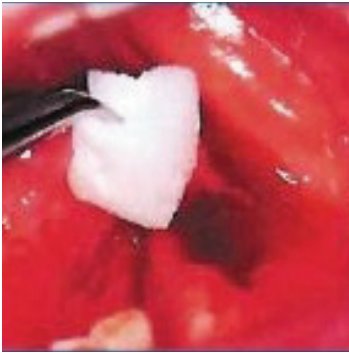

TOOTH EXTRACTIONS

Oozing bleeding from alveolus is stopped by application of SURGISPON® Dental cube.

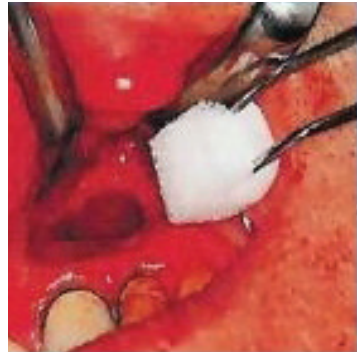


ENUCLATION OF CYSTS

Bleeding from the cavity is controlled with the dental cube.



DETAIL

Dental cube place in cavity.

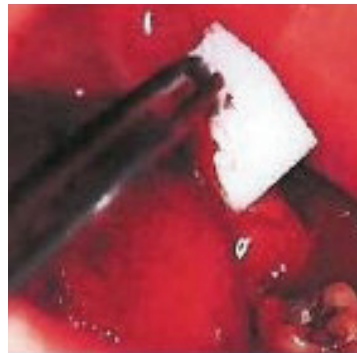

CONTROL OF PROFUSE

Bleeding after tooth extraction by means of several dental cubes

Ordering Information



SSP 101010.12 - CUBE
SSP 101010.32 - CUBE



SSP 1208 - TAMPON



SSP 202007 - SHEET

Product Code	Product	Dimensions (mm)	No. of Pcs. Per Box
SSP 101010.12	CUBE	10x10x10	12
SSP 101010.32	CUBE	10x10x10	32
SSP 1208	TAMPON	12x08 Ø	32
SSP 202007	SHEET	20x20x07	20



AEGIS LIFESCIENCES
Protection Through Perfection

ISO 9001, ISO 13485 & GMP CERTIFIED
215/216, Mahagujarat Estate, Gam : Moraiya,
Tal. Sanand, Dist.: Ahmedabad-382213, INDIA.
Fax :+91 2717 294295, Ph.:+91 2717 294636,
Email: sales@aegis-lifesciences.com
Web.: www.aegis-lifesciences.com

